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DEHAS Medical Systems GmbH
Wesloer Straße 112
23568 Lübeck
www.DEHAS.de

12/2021

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Minhua Medical
Qingpu District, Shanghai, No.
1591, Zhulu West Road,
Xujing Town, 789 Wufang Road,
Wujiang Town, Suzhou City

Instructions for use QUALITY NIBP Blood pressure cuffs

Quality NIBP cuffs can be used with any 1- or 2- Hose measurement system that works based on oszillimetry

1. Put the NIBP Hose Connector into the NIBP Input Connector on the measuring device.
2. Make sure that you always use the right cuff size for your patient, like printed on the cuff. If the cuff is too small for the patient extremities, it can be cause failures at the shown measurement figures on the measuring device.
3. During the period of use the cuffs can be become softer, this can cause crinkles. This crinkles can make dents into the patient skin. Cuffs that are too soft should be replaced.
4. Make sure that NIBP hoses not blocked or squeezed.
5. Put the cuff above the elbow (or on the knee with the right sized cuff) with the hook and loop fastener upside. Because the cuff is without a microphone, a specific position doesn't matter. Close the cuff. Make sure that the cuff not skid, but doesn't pressurises the veins & arteries.
6. Check the good circulation of the extremity but make sure that the extremity not get moved.
7. On the extremity that is used for the NIBP measurement, you may not measure SpO2, because this can cause measurement failures for the SpO2 parameter.
8. Cleaning the cuff is possible with soft disinfectant or you can wash the cuff with the 30°C programme in your washing machine.

The following cuffs are available

Cuffs for Patient monitors:

Single hose:

Quality NIBP 1 OS-* Femoral	42-50 cm
Quality NIBP 1 ERL-* Adult Long	32-42 cm
Quality NIBP 1 ER-* Adult standard	26-35 cm
Quality NIBP 1 KG-* Children, big	18-26 cm
Quality NIBP 1 K-* Children, standard	13-20 cm
Quality NIBP 1 KK-* Children, small	8-14 cm

Double hose:

Quality NIBP 2 OS-* Femoral	42-50 cm
Quality NIBP 2 ERL-* Adult long	32-42 cm
Quality NIBP 2 ER-* Adult, standard	26-35 cm
Quality NIBP 2 KG-* Children, big	18-26 cm
Quality NIBP 2 K-* Children, standard	13-20 cm
Quality NIBP 2 KK-* Children, small	8-14 cm

Cuffs for Long Term NIBP measurement:

Quality NIBP 1 ERL Adult long	32-42 cm
Quality NIBP 1 ER Adult, standard	26-35 cm
Quality NIBP 1 KG Children, big	18-26 cm
Quality NIBP 1 K Children, standard	13-20 cm
Quality NIBP 1 KK Children, small	8-14 cm

Monitor manufacturer / Adapters

Datex =*DT
 Dinamap / Critikon =*DM
 Hellige / Dräger =*LL
 Marquette / GE =*MQ
 Rectus coupling =*R

Cuffs for sphygmomanometers:

Quality NIBP 1 ERL BDM Adult, long	32-42 cm
Quality NIBP 1 ER BDM Adult, standard	26-35 cm
Quality NIBP 1 KG BDM Children, big	18-26 cm

Quality NIBP 2 ERL BDM Adult long	32-42 cm
Quality NIBP 2 ER BDM Adult standard	26-35 cm
Quality NIBP 2 KG BDM Children, big	18-26 cm

Cuff replacement bladders:

Quality NIBP 1 ERL BLA Adult, long	32-42 cm
Quality NIBP 1 ER BLA Adult standard	26-35 cm
Quality NIBP 1 KG BLA Children, big	18-26 cm

Quality NIBP 2 ERL BLA Adult, long	32-42 cm
Quality NIBP 2 ER BLA Adult standard	26-35 cm
Quality NIBP 2 KG BLA Children, big	18-26 cm

Distributed by:

DEHAS Medical Systems GmbH
 Wesloer Straße 112
 23568 Lübeck
 www.DEHAS.de 12/2021

Constructed by:

Minhua Medical
 Qingpu District, Shanghai, No.
 1591, Zhulu West Road,
 Xujing Town, 789 Wufang Road,
 Wujiang Town, Suzhou City